ESPAÑOL

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
### Adult Instructions for use

#### PURPOSE


This chamber is designed for use with an inhaler that delivers aerosolized medication to the lungs, following medical indications.

#### HOW TO USE THE CHAMBER

The chamber can be used as soon as the packaging is removed. Before use carefully read these instructions as well as those supplied with the inhaler.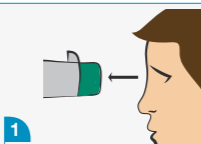


**1** Before use, carefully examine the chamber. Replace it or contact your pharmacist if you see any damaged parts.

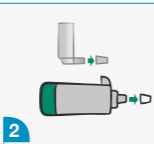
**2** Remove the caps from the inhaler and the chamber.


**3** Shake the inhaler before each use as per the instructions supplied by the manufacturer.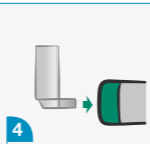

**4** Insert the inhaler into the grey-coloured backpiece of the chamber.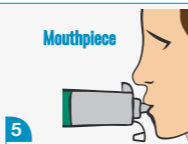

**5** **Mouthpiece:** Put the mouthpiece into your mouth and close your lips around it, to ensure a tight seal. The flow signal only moves if the lips are tightly sealed around the mouthpiece. **Large Mask:** Place the mask on the face ensuring a tight seal between the mask and face. The flow signal only moves if there is a tight seal.

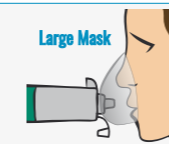**6** **Mouthpiece:** Breathe out gently and then press down on the inhaler at the start of a slow inward breath. Use the flow signal to help coordinate this movement. Breathe in slowly and deeply through your mouth, until your lungs are full. Hold your breath for 5-10 seconds, if possible, and breathe out. If you cannot hold your breath for at least 10 seconds, then keep your lips tight around the mouthpiece and, after you have pressed the inhaler, breathe in and out normally 2 or 3 times into the chamber. If you hear the sound of the whistle you are breathing in too fast. So slowly breathe in. Spray only one puff at a time when you need to inhale. **Large Mask:** Breathe out gently and press down on the inhaler at the start of a slow inward breath. Use the flow signal to help coordinate this movement. Keep the mask placed tightly on the face while breathing 5 or 6 times after you have pressed the inhaler. If you hear the sound of the whistle you are breathing in too fast. So slowly breathe in. Spray only one puff at a time when you need to inhale.

**7** Follow the instructions for use on your inhaler to know how long to wait before repeating steps 3-6.

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ENGLISH

### CLEANING INSTRUCTIONS

The chamber can be used directly after remove the packaging. It should be cleaned weekly.

- 1 Remove the backpiece. Remove the mouthpiece cap (if applicable) or the mask.
- 2 Soak the parts for 20 minutes, in clean, warm, soapy water. Shake gently. Rinse the parts with clean water.
- 3 Shake off excess water and allow to air dry upright. Make sure all the parts are dry before putting back together.
- 4 Press firmly to attach the backpiece.

### NOTES

- Replace this product after 12 months from first use.
- This product does not contain latex.
- Do not share this device.
- If you notice medication build-up in the chamber, wash the inside of the chamber gently with a cloth.

### ⚠ PRECAUTIONS

- Do not leave the chamber unattended with children. It is not a toy.
- The product can be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

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