ESPAÑOL

Neonatal/Pediátrico

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
Instructions for use

PURPOSE


This chamber is designed for use with an inhaler that delivers aerosolized medication to the lungs, following the medical indications.

HOW TO USE THE CHAMBER

The chamber can be used directly after removing the packaging. Before use carefully read these instructions as well as those supplied with the inhaler.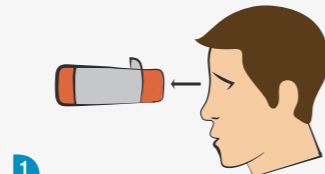


1 Before use, carefully examine the chamber. Replace it or contact your pharmacist if you see any damaged parts.

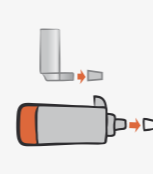
2 Remove the caps from the inhaler and the chamber.


3 Shake the inhaler before each use as per the instructions supplied by the manufacturer.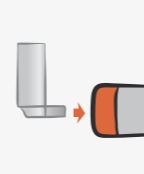

4 Insert the inhaler into the grey-coloured backpiece of the chamber.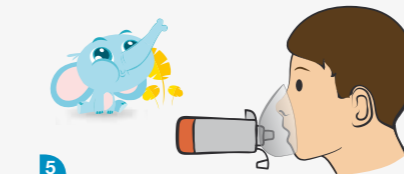

5 Place the mask on the face to ensure a tight seal. The flow signal only moves if the mask is snug to the child's face.

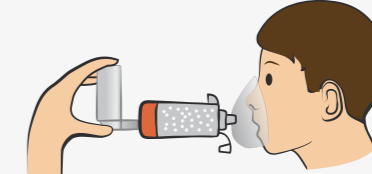6 Press down on the inhaler at the start of a slow inward breath, while the flow signal moves towards the child. Keep the mask firmly on the face for 5-6 breaths after pressing the inhaler. Spray only one puff at a time.

7 Follow the instructions for use on your inhaler to know how long to wait before repeating steps 3-6.

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
ENGLISH

CLEANING INSTRUCTIONS

The chamber can be used directly after remove the packaging. It should be cleaned weekly.

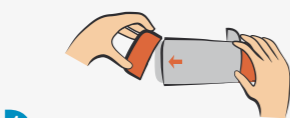1
Remove the backpiece. Remove the mask.

2
Soak the parts for 20 minutes, in clean, warm, soapy water. Shake gently. Rinse the parts with clean water.

3
Shake off excess water and allow to air dry upright. Make sure all the parts are dry before putting back together.

4
Adjust the mask.

NOTES

- Replace this product after 12 months from first use.
- This product does not contain latex.
- Do not share this device.
- If you notice medication build-up in the chamber, wash the inside of the chamber gently with a cloth.

PRECAUTIONS

- Do not leave the chamber unattended with children. It is not a toy.
- The product can be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

TIPS

- Give positive encouragement to the child.
- Look at our pet and have the child use it like an elephant's trunk, or use a teddy bear or a doll for the child to see it as a game.
- If possible, give the treatment to your child when he/she is happy and not crying or upset.

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